


# WINBEX<sup>®</sup> Saddle Stool

# ASCOT Instructions

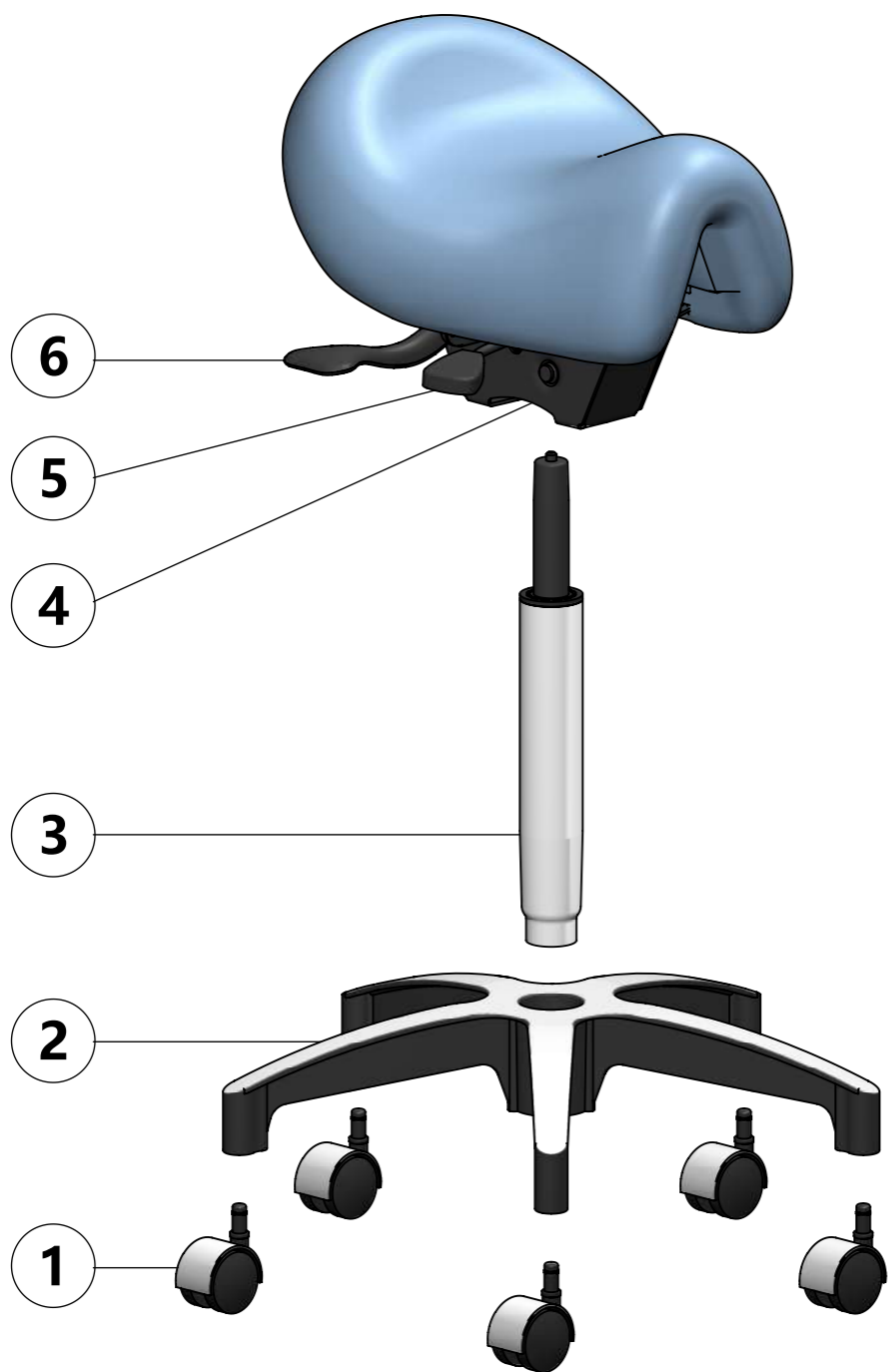

This stool requires a limited amount of assembly.  
Please read the instructions enclosed:

## Assembly Instructions

1. Attach wheels (1) onto base (2)
2. Insert gas cylinder (3) into base (2)
3. Place seat (4) (with mechanism assembled) onto gas cylinder (3)

## Stool Function Adjustments

### Seat Height

The height of the seat can be adjusted by pulling the seat height lever (5) upwards and releasing to desired height while the occupant takes their weight off the stool.

### Seat Tilt

The seat has 13 degrees of forward and 9 degrees of rearward tilt.

Push the handle (6) downwards to "free float" the movement and lift upwards to lock in any position.

## WINBEX CORPORATION

95, Nangang 3rd Rd., Nangang Industrial Park,  
Nantou City 54067, Taiwan

T/ +886-49-2260938 F/ +886-49-2260431  
E/ winbex@winbex.com.tw W/ winbex.com